


## SETTING THE BEL-20 PRESET AND AUTO-RETURN FUNCTIONS

The Bel-20 chair has 1 programmable Preset function (P button) and 1 programmable Auto-Return function (A button) located on the foot control.

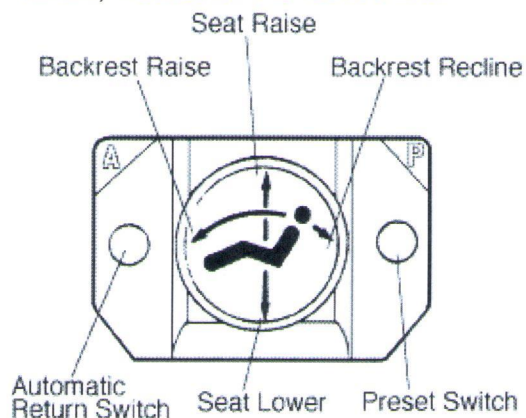

BEL-20 FOOT CONTROL

1. To program the **Preset** function set the chair backrest and base to the desired position.
2. Once the back and base are set, press and hold the P button on the foot control until an audible tone sounds.
3. When the audible tone sounds, the chair is now programmed. Test the chair.
  
1. To program the **Auto-Return** (patient exit position) function set the chair backrest and base down to an acceptable patient exit position.
2. Once the back and base are set, press and hold the A button on the foot control until an audible tone sounds.
3. When the audible tone sounds, the chair is now programmed. Test the chair.

*Note: On older model chairs the audible tone may not sound. In this case hold the applicable button for 5 to 7 seconds. Test the chair.*