




Film Processor Maintenance

Consistent, artifact-free film processing is invaluable to your practice. You rely on diagnostic quality radiographic films for diagnosis, presentation of treatment plans, and patient education.

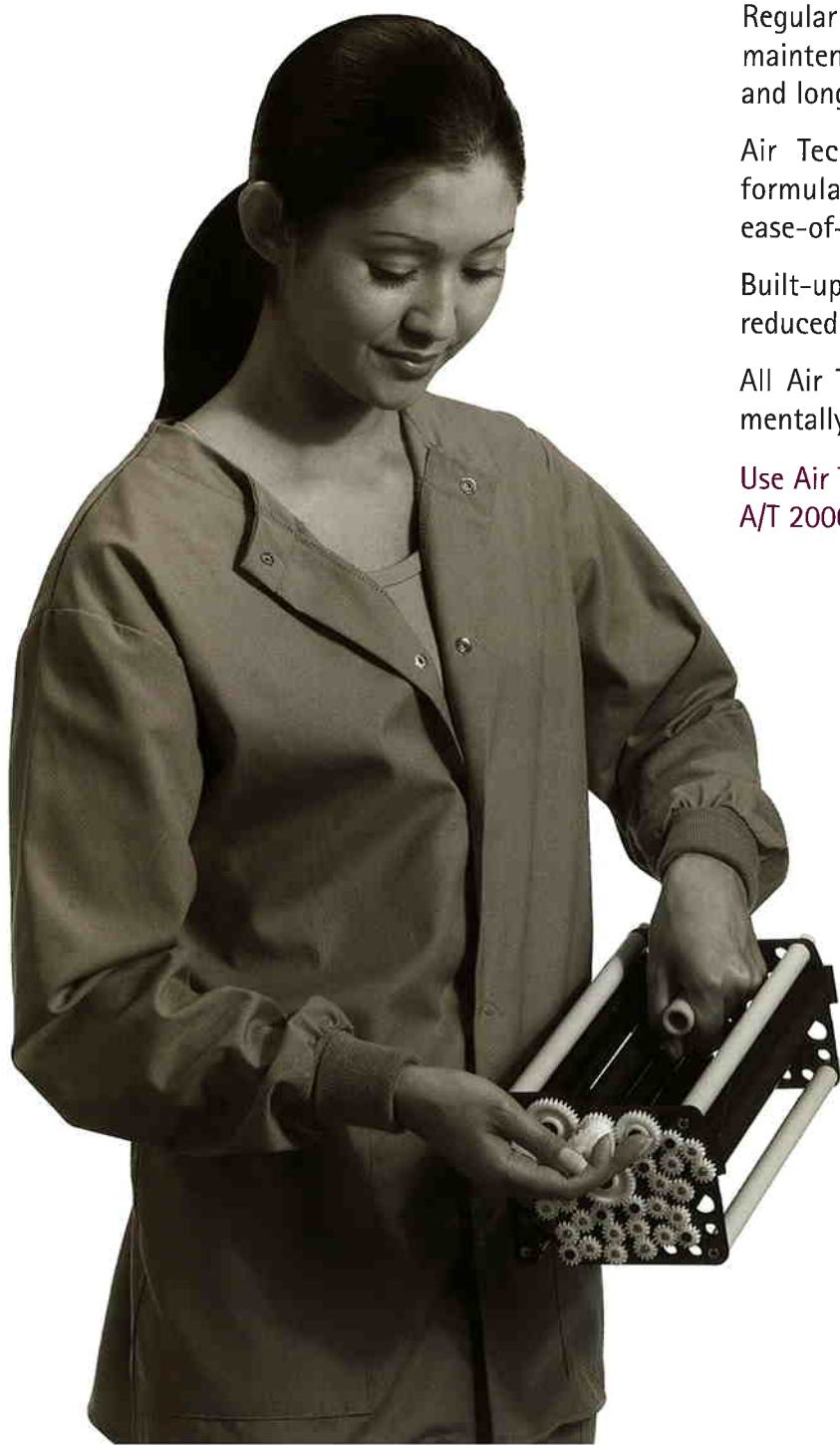

Regular maintenance of your processor with the right maintenance products ensures best quality radiographs and longest service life.

Air Techniques' maintenance products are specially formulated for maximum cleaning efficiency and ease-of-use.

Built-up residues are easily removed and staining is reduced; protecting films from roller deposits.

All Air Techniques' maintenance products are environmentally safe, they do not contain any heavy metals.

Use Air Techniques' maintenance products for A/T 2000, A/T 2000Plus, A/T 2000XR and all models of Peri-Pro.

**AIR
TECHNIQUES**
INC.

Maintenance for Roller Transport Processors



Pour Component 1 of Formula 2000 in Tank.



Add water and Component 2.



Insert roller transport.

Quarterly Cleaning with Formula 2000

Formula 2000 is used for quarterly cleaning to reduce staining and prolong the life of your processor. Formula 2000 is a remarkable two-component formulation. It contains no dichromates or other heavy metals. The deep cleaning of your Roller Transport Processors with Formula 2000 takes only 15 minutes. It dissolves deposits, dirt and algae which are residues of film processing.

Weekly Cleaning with Spray 2000

Used for weekly cleaning of roller processors and dip tanks to remove residue, protect films from roller deposits and improve the week-to-week performance of your processor.

Spray 2000 is available in a convenient, ready to use spray bottle. No mixing necessary, just spray, sponge and rinse under running water. Use Spray 2000 for A/T 2000, A/T 2000Plus and A/T 2000XR



Spray 2000



Formula 2000

Maintenance for Peri-Pro Transports



Add cold water and both powder packages of Formula 2000. Soak for 30 minutes.



Add both bottles Formula 2000 Component 1.



Rinse transport and place into cleaning tub.

Peri-Pro Transport Cleaning Kit

Used for quarterly cleaning of Peri-Pro Transports to reduce staining, decrease normal maintenance time and prolong the life of your Peri-Pro. Peri-Pro Transport Cleaning Kit is an easy and convenient way to care for your Peri-Pro. It is specially designed to hold the transport with just the right amount of Formula 2000 and water. The tub is completely self-contained, portable and requires little counter space. The deep cleaning with Peri-Pro Transport Cleaning Kit takes only 30 minutes.



Peri-Pro Cleaning Kit